

PATGuard 3 – Installing on a Network Workstation


Installing on a Network Workstation

To install PATGuard 3 on a network Workstation, browse to the location specified when you installed on a network Server (See How to Guide 047- Installing on a Network Server). Locate and run the file **netsetup.exe** to install PATGuard 3 on a network Workstation. Click **Next >** on the following screen to start the installation process.



Installation Screen 1

Review the [Licence Agreement](#) and click **Next >** to continue.

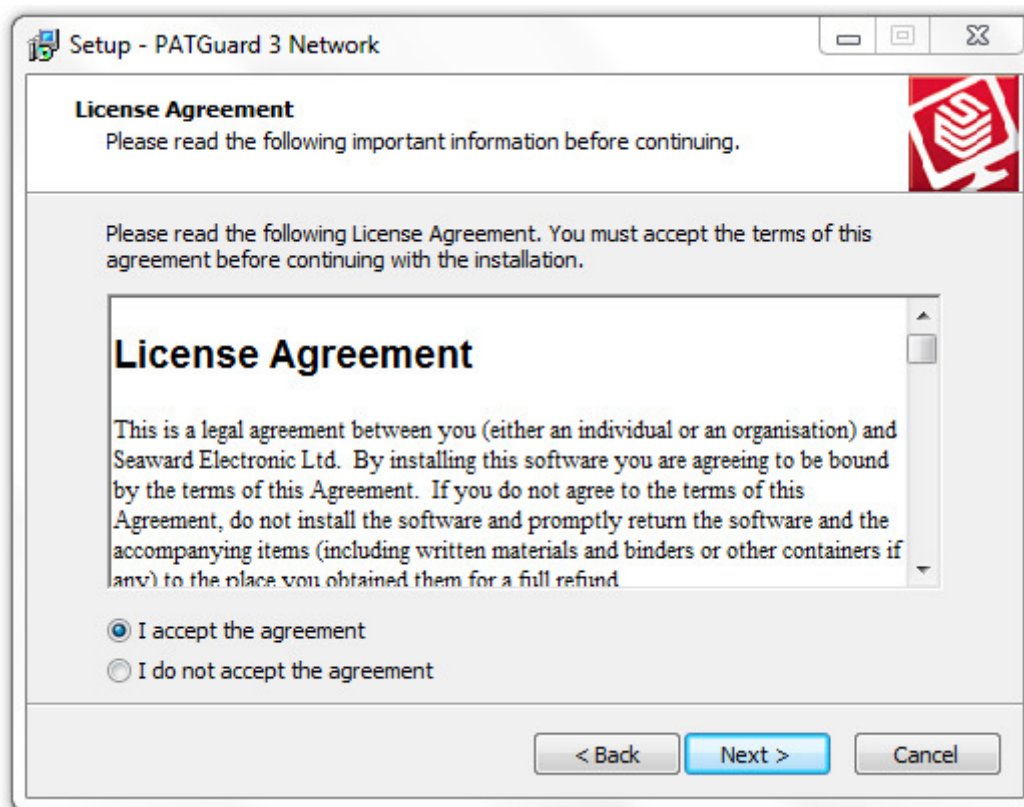

Follow the remaining on-screen instructions to install PATGuard 3 on your network Workstation.

If you encounter problems installing PATGuard 3, please refer to the [Support](#) page.